

OraStim

ORAL STIMULATOR



A dedicated tool for the therapist to encourage stimulation in the oral cavity

OraStim can be used to stimulate the faucal arches to improve swallow reflex.

...dedicated to effective rehabilitation




KAPITEX

Adding Quality of Life to
Airway Management

OraStim

ORAL STIMULATOR

A dedicated tool for the therapist to encourage stimulation in the oral cavity following surgery, stroke or other neurological disorders.

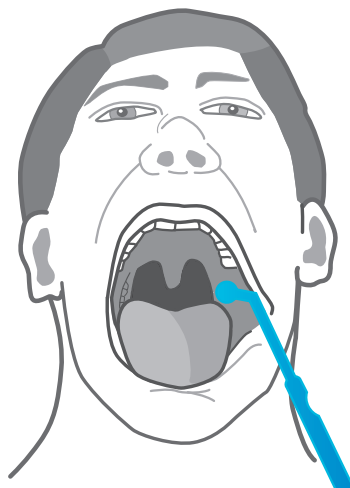

OraStim is a therapy tool that Speech Therapists can use in Dysphagia Therapy to increase the sensitivity of the faucal pillars/arches, base of tongue and margin of velum - through thermal, tactile and sensory stimulation (dipping in lemon juice), to help initiate a swallow reflex.

OraStim has two heads, one smooth and one textured. Both ends are suitable to assess the sensitivity of

the oral cavity. The therapist should always start giving therapy with the smooth end and move onto the textured end if appropriate. The OraStim can be bent/shaped to enable easy access to the oral cavity.

OraStim can be used to stimulate the **fauca arch** to improve the swallow reflex.

OraStim can be used as a **taste tester** by dipping either end into any preferred testing material.

OraStim can be used for **thermal stimulation** by dipping either end into iced water. Please note that the OraStim is made of plastic material and does not have the same cooling/thermal properties as metal.

Cleaning Instructions

OraStim can be cleaned in a domestic dishwasher or in soapy water. Rinse in clean water before use.

Please Note: All patients must be assessed by a Speech and Language Therapist or other qualified clinician to ensure OraStim is suitable.

Ordering Information

OS ORS 0001 OraStim Oral Stimulator - Pack of 10



Smooth head



Textured head

OraStim can be bent/shaped to enable easy access to the oral cavity

